Vectra[®]

Guiding Rheumatoid Arthritis
Treatment Decisions


A Patient Guide to Vectra[®]

Vectra Measures Rheumatoid Arthritis Inflammation

Rheumatoid arthritis (RA) is a complicated disease with symptoms that can vary widely. Vectra reveals the extent of RA inflammation inside your body.

FACT:

Rheumatoid arthritis inflammation can damage joints and organs¹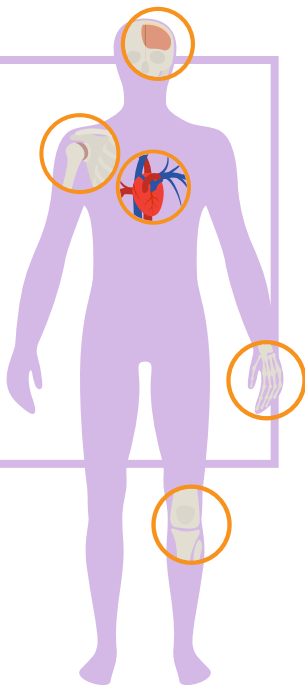

RA is more than pain and joint damage.


RA inflammation can damage organs like the heart and put you at risk for cardiovascular disease.² The goal of treatment is to protect you from damage caused by RA inflammation.

Vectra helps by providing an objective, molecular assessment of your body's RA inflammation to complement your reported symptoms and your doctor's assessment. By using the Vectra test over time, your Vectra Scores reveal how well your current treatment is controlling inflammation.³

Vectra is the Best Predictor of Future Joint Damage⁴

FACT: 42% of patients thought to be in low or moderate disease activity by traditional disease activity measure were in high disease activity measured by Vectra and thus at risk for future joint damage^{5,6}

Vectra is an advanced blood test that objectively measures inflammation caused by rheumatoid arthritis. Vectra measures 12 biomarkers implicated in RA disease. Other tests, such as C-reactive protein (CRP) and erythrocyte sedimentation rate (ESR or "sed rate"), only measure one marker. No single biomarker has been proven reliable for predicting future joint damage. **Vectra is the best predictor of your risk for developing new damage to your joints.**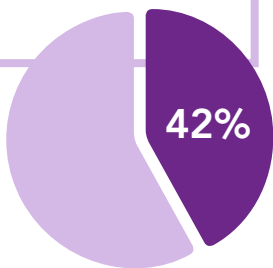

Vectra Reveals How Your Current Treatment is Working


If your Vectra Score is in the low range, this indicates a low level of RA inflammation. If your Vectra Score is in a moderate or high disease activity category, your doctor may recommend modifying your treatment plan to try to improve your response. An improved response would mean a decrease in your disease activity and also a lower risk of joint damage.

Understanding Your Vectra Score

Vectra measures 12 biomarkers found in your blood that are implicated in rheumatoid arthritis and affect the level of RA inflammation. The scores range from 1-100 with lower scores indicating less RA inflammation and higher scores indicating more inflammation. Patients with Vectra Scores in the low range are at a lower risk of future joint damage.

The Vectra Score is reported on a **scale of 1-100**.

Patients in the high and moderate categories are considered to have uncontrolled inflammation and may require treatment modification.

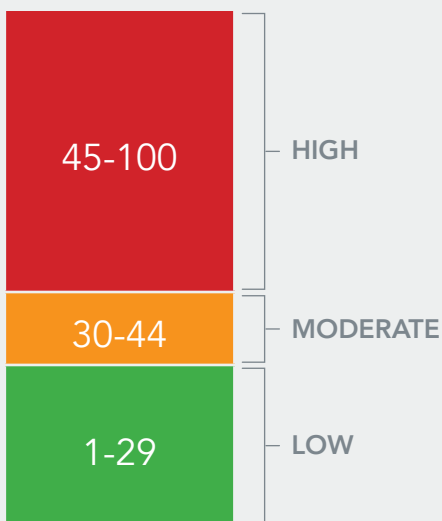The goal is low disease activity

Vectra Scores over time can help you and your doctor evaluate how well your treatment plan is controlling your RA inflammation.

Vectra SAMPLE

ORDERING PHYSICIAN: Physician Name, MD		
RECEIVING HEALTHCARE PROVIDER Clinic: Clinic United Partnership Example of Oklahoma University Phone: 555-555-1234 Fax: 555-555-1234 Report Date: FEB-01-2019	SPECIMEN Collection Date: JAN-01-2019 Receipt Date: JAN-07-2019	PATIENT Name: Jane Doe Date of Birth: APR-17-1980 Patient ID: 000-000-0000 Gender: Female TRF ID: 00000000-00

Vectra Molecular Result

Vectra Score	Change in Score	Vectra Score Interpretation
48 HIGH (45-100)	↑ Meaningful Increase ≥8 Units	<p>■ High Vectra Score: 48</p> <p>Patient has a High Vectra Score that increased by 8 or more units from the previous score, which was in the Moderate category. The patient is at increased risk for radiographic progression. Consider adjusting treatment regimen to reduce inflammation, and retesting at the next clinical visit.</p>

VECTRA SCORE DESCRIPTION

Vectra Disease Activity Levels: Low: 1 to 29 Moderate: 30 to 44 High: 45 to 100

Vectra measures the concentrations of 12 serum proteins. An algorithm is applied to these concentrations to calculate a quantitative disease activity score on a scale of 1 to 100. The Vectra score is adjusted based on the age, gender, and adiposity of the patient. Test results are intended to aid in the assessment of disease activity in patients with rheumatoid arthritis (RA) when used in conjunction with standard clinical assessment. This test is not intended or validated to diagnose RA.

CHANGE IN SCORE DESCRIPTION

Change in score is related to the Minimally Important Difference (MID) for Vectra. The MID is the smallest absolute change in Vectra score that is likely to reflect change in disease activity and not be due only to random variation. The MID for patients with a moderate or high Vectra score is 8.0. This MID does not apply to patients with a low Vectra score. Treatment management recommendations should be interpreted in the context of all available clinical information.

Vectra Scores Over Time Complete score history shown on next page.

VECTRA SCORE INTERPRETATION DESCRIPTION

The interpretation of the Vectra score in relation to the risk of radiographic progression and level of inflammatory disease activity is more detailed and up-to-date information before developing a treatment plan for a particular patient. While the Vectra score provides important objective data, it is intended for informational purposes only and does not constitute a recommendation. Medical management decisions should be made by a healthcare provider with an understanding of the full medical history and clinical assessment of the patient.

Cicondo Bioscience, Inc. a Myriad Genetics Company
300 Wilkens Way, Salt Lake City, UT 84103
Web: cicondo.com | Fax: 1-877-743-8460 | TOLL-FREE: 1-877-743-8629

The Vectra test is intended for clinical use and determined its performance is certified under the Clinical Laboratory Improvement Act of 1988 (CLIA) as qualified to perform high complexity clinical testing and is a California Laboratory Director Bruce F Arnold, MD, FACP

Cicondo Bioscience, Inc. and Myriad Genetics, Inc. developed and validated the Vectra test. The Cicondo Bioscience Clinical Laboratory is a CLIA-certified laboratory. Cicondo Bioscience, Inc. is a California Laboratory Director Bruce F Arnold, MD, FACP. Cicondo Bioscience, Inc. is a Myriad Genetics Company. Cicondo Bioscience, Inc. is a California Laboratory Director Bruce F Arnold, MD, FACP.

PR 2018EV 2 / 4 19 Laboratory Director: Bruce F Arnold, MD, FACP 1 of 3

Meaningful Change of ≥8 Units in High or Moderate Vectra Scores. If your score goes up or down by 8 or more units, that means your level of RA inflammation has changed.⁷

Your Vectra Score is personalized for you. Because people tend to have higher or lower levels of inflammation based on their age, gender, and level of body fat, each patient's score is adjusted to account for this. This means that your score accurately reflects the RA inflammation you are experiencing.

You and your doctor don't need to rely on symptoms alone. Vectra can be used to help guide treatment decisions to take control of your disease.

Steps to Getting Vectra

Step 1

Your doctor orders a Vectra test for you

Step 2

One tube of blood is collected at your doctor's office, a local lab, a LabCorp® patient service center*, or your home by a certified professional

Step 3

Your blood sample is sent to our lab for analysis

Step 4

Your doctor receives your Vectra test report in approximately 5-7 calendar days

Step 5

Access your test result and see your score at VectraScore.com or at your next doctor appointment

*LabCorp contracts do not apply as Myriad will perform the test and bill for Vectra

Viewing Your Results

Approximately 10 days after getting your blood drawn, you can directly access your Vectra test report online by going to [VectraScore.com](https://www.vectrascore.com) and clicking on the [myVectra™](#) link. You can also contact Vectra customer service team at 1 (877) 743-8639 to obtain a copy of your test report.

10
Days

GO TO

[VectraScore.com](https://www.vectrascore.com)

OR CALL CUSTOMER SERVICE AT

1-877-743-8639

FOR PATIENT MEDICAL SUPPORT CALL

1-877-743-8639 (ext. 1984)

OR EMAIL

VectraMed@myriad.com

Vectra Complements Physician & Patient Assessment

Disease Management is a Shared Effort

The most comprehensive rheumatoid arthritis assessment combines:

Patient
Assessment

Physician
Assessment


Molecular
Assessment

Vectra[®]

molecular assessment
of 12 biomarkers

Your Vectra Score, along with how you are feeling and your doctor's expertise, can help guide treatment for the best outcome possible for you.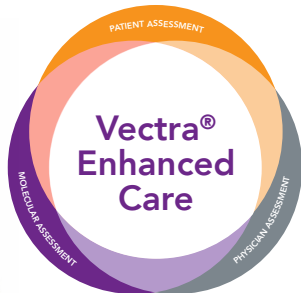

1. Centers for Disease Control and Prevention. Rheumatoid Arthritis. www.cdc.gov/arthritis/basics/rheumatoid-arthritis.html. Accessed 2/15/19. 2. Arthritis Foundation: How Arthritis Inflammation Affects Your Heart. www.arthritis.org/living-with-arthritis/comorbidities/heart-disease/ra-and-your-heart.php. Accessed 6/4/19. 3. Roodenrijs NMT, et al Arthritis Res Ther. 2018 Nov 20;20(1):256. doi: 10.1186/s13075-018-1750-5. 4. Ann Rheum Dis. 2015 Jun;74(6):1102-9. Epub 2014 May Ann Rheum Dis. 2015 Jun;74(6):1102-9. Epub 2014 May 8. 5. Data on file, Crescendo Biosciences, for a combined analysis of 147 patients with low or moderate DAS28-CRP from the Leiden, OPERA and SWEFOT studies. 6. Curtis JR, et al. Adjustment of the multi-biomarker disease activity score to account for age, sex, and adiposity in patients with rheumatoid arthritis. Oxford Academic, Rheumatology. 2018; Dec 24. 7. Chernoff D, et al. Determination of the minimally important difference (MID) in multi-biomarker disease activity (MBDA) test scores: impact of diurnal and daily biomarker variation patterns on MBDA scores. Clin Rheumatol. 2018; Aug 29.

Cost & Financial Assistance

Simple Billing, No Surprises

We are committed to making precision testing accessible to RA patients.

Affordability. Due to insurance coverage and our Vectra CARE financial assistance plan:

- Over 95% of patients owe \$90 or less
- Up to 70% of patients will owe \$0

Certainty. If the out of pocket is estimated to be more than \$90, Vectra Customer Service will call you before running the test.

Financial Assistance. Our CARE Financial Assistance program can reduce the Patient Responsibility to as low as \$0 depending on your household annual income.

Should you encounter **ANY** financial hardship associated with your bill, Vectra Billing will work directly with you.

Apply for financial assistance by calling 1-877-743-8639 or visiting [VectraScore.com](https://www.VectraScore.com)

Notice and Statement Concerning Nondiscrimination and Accessibility

Discrimination is Against the Law

Myriad complies with applicable Federal civil rights laws and does not discriminate on the basis of race, color, national origin, age, disability, or sex. Myriad does not exclude people or treat them differently because of race, color, national origin, age, disability, or sex.

Aids and Services

Myriad provides free aids and services to people with disabilities to communicate effectively with us, such as TTY/TDD calls or written information in suitable formats. Myriad will also provide free language services to people whose primary language is not English through qualified interpreters.

If you need these services, contact Mr. Don Martin:

Don Martin
Compliance Director
320 Wakara Way
Salt Lake City, UT 84108
Telephone: (801) 584-3600
Fax: (801) 883-3472
Email: compliance@myriad.com

Grievances

If you believe that Myriad has failed to provide these services or discriminated in another way on the basis of race, color, national origin, age, disability, or sex. You can file a grievance by mail, telephone, fax, or email. If you need help filing a grievance, Mr. Martin is available to help you (see contact information above).

Grievance Procedure

1. Any person who believes someone has been subjected to discrimination by Myriad on the basis of race, color, national origin, sex, age or disability may file a grievance with Myriad. It is against the law for Myriad to retaliate against anyone who opposes discrimination, files a grievance, or participates in the investigation of a grievance.
2. Grievances must be submitted within 60 days of the date the person filing the grievance becomes aware of the alleged discriminatory action.
3. The complaint must be in writing, containing the name and address of the person filing it. The complaint must state the problem or action alleged to be discriminatory and the remedy or relief sought.
4. Myriad will conduct an investigation of the complaint. This investigation may be informal, but it will be thorough, affording all interested persons an opportunity to submit evidence relevant to the complaint. Myriad will maintain the files and records relating to such grievances pursuant to its document retention policies. To the extent possible, and in accordance with applicable law, Myriad will take appropriate steps to preserve the confidentiality of files and records relating to grievances and will share them only with those who have a need to know.
5. Myriad will issue a written decision on the grievance, based on a preponderance of the evidence, no later than 30 days after its filing, including a notice to the complainant of their right to pursue further administrative or legal remedies.
6. The person filing the grievance may appeal Myriad's decision in writing to the President of Myriad within 15 days of receiving Myriad's initial decision. The President will issue a written decision in response to the appeal no later than 30 days after its filing.
7. Individuals seeking access to Section 1557 and its implementing regulations may be facilitated by contacting Mr. Martin (see contact information above).
8. The availability and use of this grievance procedure does not prevent a person from pursuing other legal or administrative remedies, including filing a complaint of discrimination on the basis of race, color, national origin, sex, age or disability in court or with the U.S. Department of Health and Human Services, Office for Civil Rights. A person can file a complaint of discrimination electronically through the Office for Civil Rights Complaint Portal, which is available at: <https://ocrportal.hhs.gov/ocr/portal/lobby.jsf>, or by mail or phone at:

U.S. Department of Health and Human Services
200 Independence Avenue, SW
Room 509F, HHH Building
Washington, D.C. 20201
9. Complaint forms are available at: <http://www.hhs.gov/ocr/office/file/index.html>. Such complaints must be filed within 180 days of the date of the alleged discrimination. Myriad will make appropriate arrangements to ensure that individuals with disabilities and individuals with limited English proficiency are provided auxiliary aids and services or language assistance services, respectively, if needed to participate in this grievance process. Mr. Martin will be responsible for such arrangements.

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// The Vectra Score was extremely validating and empowering. My score is unbiased and represents exactly what I'm feeling.* //

- Stephanie, Vectra Patient

Vectra[®]

VectraScore.com // 1-877-743-8639

Myriad Autoimmune
320 Wakara Way, Salt Lake City, UT 84108

*These statements reflect some patients' personal experience and may not be the view of the Company.

Vectra is validated for use in adults diagnosed with rheumatoid arthritis. Test results are intended to aid in the assessment of disease activity in RA patients when used in conjunction with standard clinical assessment. This test is not intended or validated to diagnose RA.

©2019 Myriad Genetics, Inc. All rights reserved. Vectra and the associated logo are trademarks or registered trademarks of Crescendo Bioscience Inc, an affiliate of Myriad Genetics, Inc. and its affiliates in the United States and other jurisdictions.